

## **Decontamination Checklist for PAPR**

Wearing clean gloves, use Caviwipe1(3 min), Oxivir T,b (1 min) or Sani-Cloth Prime (2min) wipes

1. First, verify the wipes effectiveness against Sars CoV 2 aka Covid 19 using this EPA website <https://www.epa.gov/pesticide-registration/list-n-disinfectants-use-against-sars-cov-2>
2. Put on disposable gloves.
3. Ensure nearby waste receptacle.
4. Detach PAPR Hood from hose
5. Detach hose from battery/filter/blower assembly.
6. Detach belt assembly from battery/filter/blower assembly.
7. Decontaminate the outer surface of the battery/filter/blower assembly with an approved decontamination wipe. Dispose of the wipe and set battery/filter/blower assembly aside into clean area.
8. Decontaminate the outside of the hose with an approved decontamination wipe. Dispose of the wipe and set the hose aside into clean area.
9. Decontaminate the outside of the PAPR hood with an approved decontamination wipe. Dispose of the wipe and set battery/filter/blower assembly aside into clean area.
10. Decontaminate the belt assembly with an approved decontamination wipe. Dispose of the wipe and set battery/filter/blower assembly aside into clean area.
11. Ensure all surfaces are exposed to the decontamination solution for the recommended contact time. You can verify the recommended contact time at the EPA website <https://www.epa.gov/pesticide-registration/list-n-disinfectants-use-against-sars-cov-2>
12. Moisten a clean cloth or towel with clean water and wipe down the outside surface of the battery/filter/blower assembly, the outside of the hose, the outside of the PAPR hood, and the belt assembly to remove residual cleansing agent. Dispose of wipes.